● ● ● **Lubrication Procedure Change** ● ● ●

■ NSK PANA SPRAY Plus

Apply NSK PANA SPRAY Plus every time after each use and/or before autoclaving.

- 1) Remove the bur from the handpiece.
- 2) Attach the Spray Nozzle into the Spray Port nozzle on the can.
- 3) Insert the Spray Nozzle in rear of the handpiece. Hold the handpiece and spray for approximately 2 - 3 seconds. Apply lubricant until it expels from the handpiece head for at least 2 seconds (Fig. 1).
- 4) Insert the Purge Nozzle into the rear of the handpiece until it is locked in place with a clicking sound (Fig. 2 ①).
- 5) Insert an air syringe/air blow gun into the Purge Nozzle. Then blow air for 30 seconds or more to purge excess oil inside the handpiece (Fig. 2 ②).

*Purge Nozzle: Order Code Z1259080

⚠ CAUTION

- Excess oil remaining inside the handpiece may cause overheating of the handpiece which could lead to a burn injury.
- Debris and blood could enter the product during surgical operations. To avoid accumulation of debris inside the product, NSK recommends lubrication with PANA SPRAY Plus immediately after use and before sterilization.
- When applying spray be sure to hold the handpiece firmly to prevent the handpiece from slipping out of the hand due to the spray pressure.
- Hold the spray can upright.

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